

Hydrofera **BLUE**®




Hydrofera Blue has unique, Non-cytotoxic, antibacterial wound dressings that address all 4 factors of Local Wound Care:

- Debridement / removes devitalized tissue
- Infection management - absorbs and kills bacteria in the dressing
- Moisture management - wicks bacteria out and away from the wound
- Flattens rolled wound edges

* CLASSIC manages all 4 factors of local wound care

Hydrofera Blue® Product Selection Guide

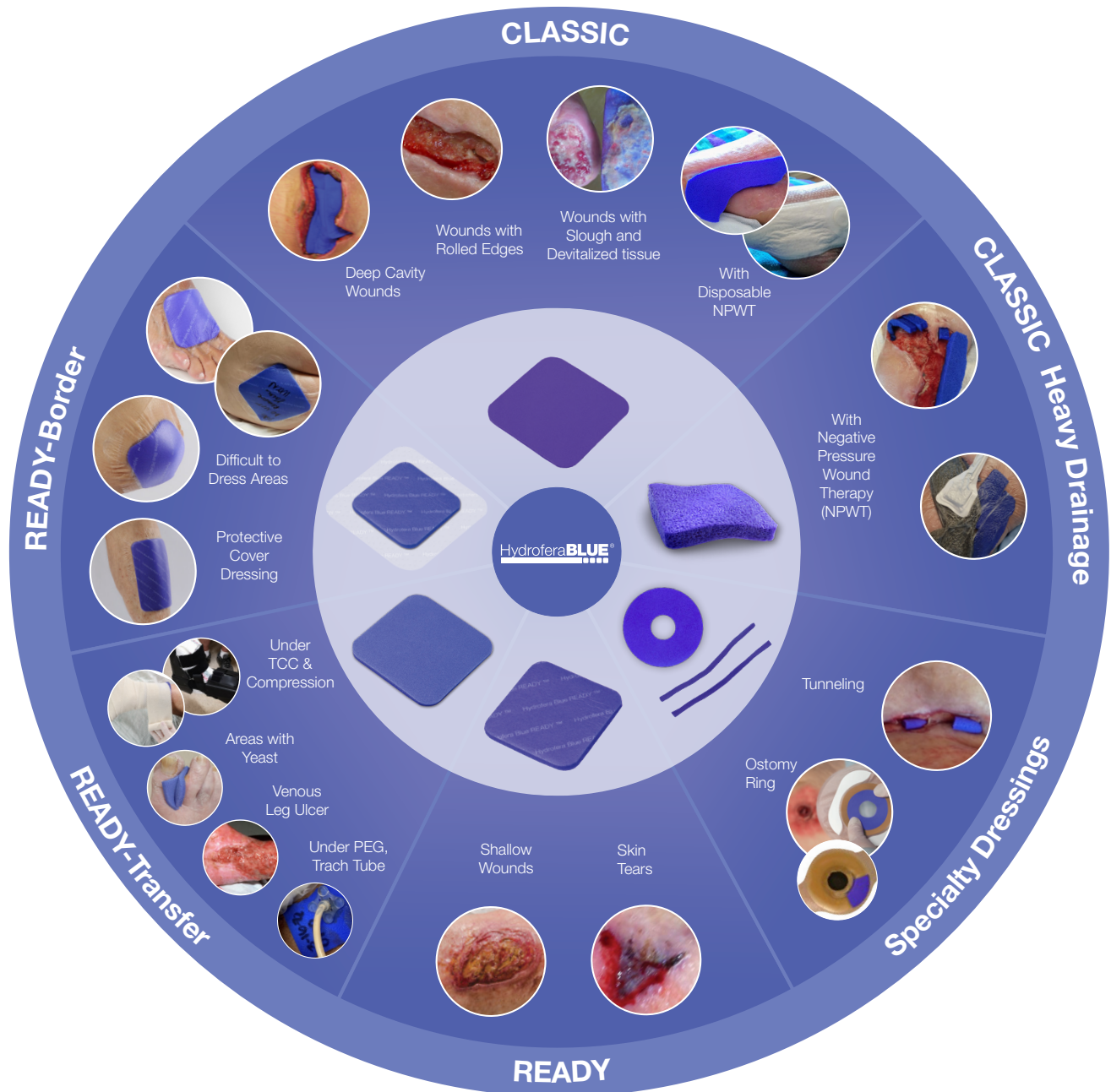

- Non-Cytotoxic
- Natural Negative Pressure*
- Natural Autolytic Debridement*

- Antibacterial
- Flattens Rolled Wound Edges*
- Disrupts Biofilm*


*CLASSIC Dressings

For product information, contact ca.customerservice@essity.com

How to Apply Hydrofera Blue Dressings

1. Prepare your supplies
2. Wash your hands
3. Remove and dispose the dressing
4. Wash your hands again
5. Clean the wound
6. Apply the new dressing


Hydrofera Blue CLASSIC®

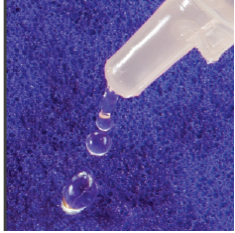


Moisten dressing with sterile saline or sterile water



Squeeze out excess

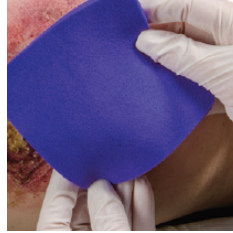

Dressing may be cut to any size. Cut ½" bigger to cover rolled edges.

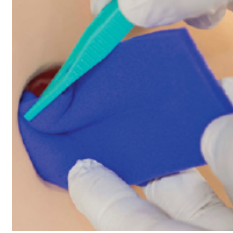


Wound with depth: loosely fill cavity, sinus or undermining.



Cover dressing: Wounds with a lot of fluid need an absorbent dressing.


Lesser fluid needs a film like cover to help keep wound moist.

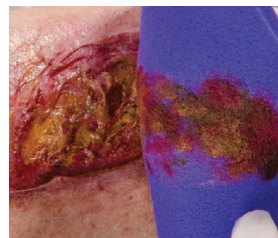

Change the dressing every 3 days. If the dressing turns white change daily.

If dressing is dry, moisten with sterile water and then remove.

Hydrofera Blue READY®

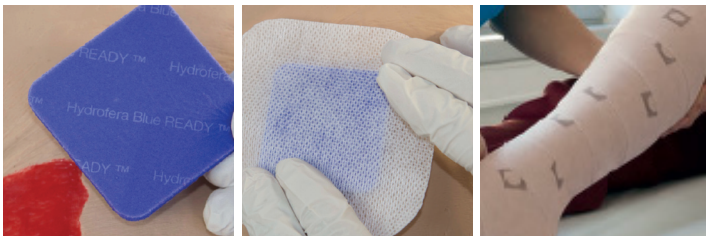

Ensure the dressing covers the entire wound. Secure with adhesive tape or fixation dressing. Can be used in conjunction with compression. Can be worn up to 7 days.


Hydrofera Blue READY®-Transfer



Ensure the dressing covers the entire wound. Apply absorptive cover dressing, like a bordered foam. Can be used with compression wrap. Can be worn for up to 7 days.

Hydrofera Blue READY-Border™

Ensure the foam pad covers the entire wound. Remove one of the liners and position dressing over the wound. Remove the second liner and smooth the dressing in place. Smooth out any bubble or wrinkles. Can be worn up to 7 days.

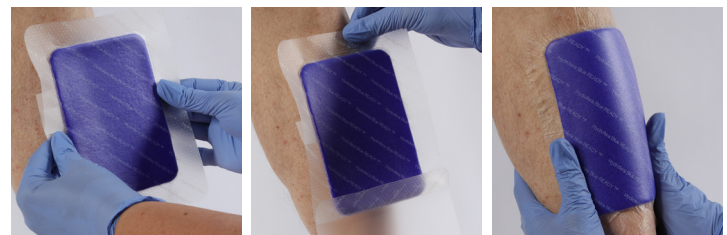

For product information, contact ca.customerservice@essity.com

Ordering Information:

Hydrofera Blue Classic antibacterial dressings



Product No Standard Dressings

HB2214
HB4414
HB6614
HBT0906

Product Size

2" x 2" (5 cm x 5 cm)
4" x 4" (10.2 cm x 10.2 cm)
6" x 6" (15.2 cm x 15.2 cm)
9-mm Tunneling Dressing (1.2 g)

Quantity/Box

10
10
10
10 pair



Heavy Drainage Dressings

HBHD4450
HBHD6675

4" x 4" x 0.5" thick (10.2 cm x 10.2 cm x 1.2 cm)
6" x 6" x 0.75" thick (15.2 cm x 15.2 cm x 1.9 cm)

5
5




Ostomy Dressing

HBRF2650

2.5" (6.4 cm) Diameter Ostomy Dressing

10

Hydrofera Blue READY antibacterial dressings

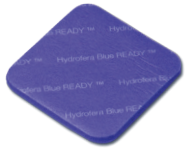

HBR2520
HBR4520
HBR8820

2.5" x 2.5" (6.4 cm x 6.4 cm)
4" x 5" (10.2 cm x 12.7 cm)
8" x 8" (20.3 cm x 20.3 cm)

10
10
10

Hydrofera Blue READY-Transfer antibacterial dressings




HBRT2525
HBRT4050
HBRT8080

2.5" x 2.5" (6.4 cm x 6.4 cm)
4" x 5" (10.2 cm x 12.7 cm)
8" x 8" (20.3 cm x 20.3 cm)

10
10
10

Hydrofera Blue READY-Border dressings

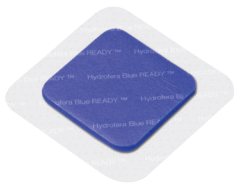

HBRB4040
HBRB5959
HBRB6080

4" x 4" (10.2 cm x 10.2 cm)
5.9" x 5.9" (15 cm x 15 cm)
6" x 8" (15 cm X 20.3 cm)

10
10
10

For product information, contact ca.customerservice@essity.com